

CONGRATULATIONS on your purchase of “GLOBAL 360”. This quality product is designed for people with special needs for wake up providing the unique feature of an Auto Repeat Countdown Timer.

BEFORE using your unit, take a few minutes to read the following instructions. The understanding of the features will allow you ease in setting and ensure effective use.

POWER SOURCE: Power Supply and Universal Adapter Plugs Included

- 1) Insert the small barrel plug of the Power Supply (*Output: DC 12V, 1000 mA; Input 100-240 Volt 50-60 Hz*) into the jack at the back of the clock labeled 12V DC Adapter.
- 2) Connect the power supply with the adapter plug that is required for your electrical outlet.
- 3) Plug into an Electrical outlet near your bedside.
*Note: 12:00 will flash on the clock display.
When you set the time, the display will stop flashing.*

BATTERY BACK UP: CR2032 Button Cell Included

In the event of a power failure, the clock will keep the clock time running and the preset times for alarm and countdown timer. The clock display will not light to save battery energy.

Note: If you have a long power failure you may need to make a slight adjustment to the clock time after the AC power returns.

To activate the battery backup function, open the Circular Battery Door located on the bottom of the clock by a turn counter clockwise and pull the protective plastic tab out.

Note: Replace Battery as required.

The life expectancy of CR2032 battery is approximately 2 years.

SETTING THE CLOCK TIME

- 1) Slide the **SET MODE** switch to “**TIME**” position.
- 2) Press and Hold the **SET** button and press the **HOUR** button to set the hours.
Note: When using the 12 Hour Clock, use the Yellow PM INDICATOR LIGHT located at the top left of the clock display for accurate setting. When the clock is reading PM time, the light will be “on”. The light will be off when the clock is reading AM time.
- 3) Press and Hold the **SET** button and press the **MINUTE** button to set the minutes.

SELECT 12 HOUR (AM/PM Time) OR 24 HOUR TIME.

- 1) Slide the **SET MODE** switch to “**TIME**” position.
- 2) Press and Hold the **SET** button and press the **START/STOP** button to change to your desired format.

SELECT THE ALERT MODE: Vibrate, Audible or Both

Slide the **ALERT MODE** switch (3-position switch located on the left side of the clock) to select the desired alerts for Clock Alarm and Countdown Timer.

VIBRATE activates bedshaker.

AUDIBLE activates sound.

BOTH activates both bedshaker and sound.

When using the **AUDIBLE** Alert, the sound can be adjusted.

Slide the **AUDIBLE** switch on the top of the clock to “**LOUD**” or “**SOFT**”.

BEDSHAKER:

When the Alarm or Countdown Timer is activated, the Bedshaker will pulse vibrate.

- 1) Insert the mini barrel plug of the Bedshaker cable into the jack at the back of the clock labeled **BEDSHAKER**, and place the Bedshaker under your pillow or mattress.
- 2) Slide the **ALERT MODE** switch (3-position switch located on the left side of the clock) to **VIBRATE** or **BOTH**.
- 3) Slide the **ALARM/COUNTDOWN TIMER** switch (4 position switch located on the right side of the clock) to **BOTH ON** position.
- 4) Press the **START/STOP** button to manually stop the vibration alert.

ALARM CLOCK

- *Alert(s) set (**VIBRATE, AUDIBLE** or **BOTH**) will be activated when the alarm time is reached.*
- *Activated Alerts can be manually stopped by pressing the **START/STOP** button while Clock display is in any **MODE**. (**TIME, ALARM** or **TIMER**) or will automatically stop after approximately 20 minutes.*
- *Set Alarm Time will remain set for daily alerts and can be turned off for days not wanted with **ALARM/COUNTDOWN TIMER** switch.*

SETTING THE ALARM TIME

- 1) Slide the **SET MODE** switch to “**ALARM**” position.
- 2) Press and Hold the **SET** button and press the **HOUR** button to set the hours.
Note: When using the 12 Hour Clock, use the Yellow PM INDICATOR LIGHT located at the top left of the clock display for accurate setting. When the display is reading PM time, the light will be “on”. The light will be off when the clock is reading AM time.
- 3) Press and Hold the **SET** button and press the **MINUTE** button to set the minutes.
*Note: The Blue **ALARM Indicator light** (located at the bottom left of the clock display) will show.
The Alarm Time will remain set and will be activated daily at the same time unless turned off or manually changed.*

TO CHECK THE ALARM TIME SETTING

Slide the **SET MODE** switch to “**ALARM**”

The Clock Display will then show the alarm time that has been set.

TURNING OFF CLOCK ALARM

Slide the **ALARM/COUNTDOWN TIMER** switch (4-position switch located on the right side of the clock) to **CLOCK ALARM OFF** or **BOTH OFF** for days you do not want the alarm clock to wake you. The Alarm time will remain set for future use.

*Note: The Blue **ALARM indicator light** will not show on the display.*

USING THE SNOOZE FEATURE

*You can press the **SNOOZE** button to stop the alarm alert and have it repeat automatically in approximately 9 minutes. This can be repeated again and again.*

After the alarm is activated, press the **SNOOZE** button to stop alert activity (audible, Bedshaker or both).

*Note: The Blue **ALARM indicator light** will flash.*

COUNTDOWN TIMER

- *Program a Countdown Time for any amount of minutes and/or hours up to 23 hours 59 minutes.*
- *Alert(s) set (**VIBRATE, AUDIBLE** or **BOTH**) will be activated when the countdown reaches “00.00”.*

- **Activated Alerts can be manually stopped by pressing the START/STOP button while Clock display is in any MODE. (TIME, ALARM or TIMER) or will automatically stop.**
Note: The auto stop feature will stop the alert(s) automatically after approximately 30 minutes if timer is set for 31+ minutes; after approximately 5 minutes if set for 6-30 minutes; and after approximately 30 seconds if set for 1-5 minutes.
- **Programmed time will automatically repeat each time the countdown reaches “00.00” unless turned off manually.**

SETTING THE COUNTDOWN TIMER:

- 1) Slide the **SET MODE** switch to “**TIMER**” position.
- 2) Press and Hold the **SET** button and press the **HOUR** button to set the hours.
- 3) Press and Hold the **SET** button and press the **MINUTE** button to set the minutes.
*Note: The Green **COUNTDOWN TIMER Indicator light** (located at the bottom right of the clock display) will show.*
- 4) Press the **START/STOP** button to start the counting down.
*Note: The Green **Countdown Timer Indicator Light** will flash. The display will show the seconds counting down when the time reaches the last minute.*

To reset the Countdown Timer,

- 1) Press the **START/STOP** button to stop the Countdown.
- 2) Press the **SET** button and the last set time will show on the display.
- 3) Reset using the **SET** button and **HOUR** and **MINUTE** buttons as noted above.

TO CHECK THE COUNTDOWN TIME SETTING:

- 1) Slide the **SET MODE** switch to “**TIMER**”. The Clock Display will show the countdown time that has been set or remains.
- 2) To check original time set, Press the **START/STOP** button and then press the **SET** button.
- 3) Press the **START/STOP** button to continue counting down the remaining time.

STOPPING REPEAT OF COUNTDOWN TIME:

- 1) Slide the **SET MODE** switch to “**TIMER**”.
- 2) Press the **START STOP** button once OR if the Alerts (Vibrate, Audible or Both) are currently activated, press the **START/STOP** button twice to stop the countdown.

TURNING OFF COUNTDOWN TIMER:

Slide the **ALARM/COUNTDOWN TIMER** switch (4-position switch located on the right side of the clock) to **COUNTDOWN TIMER OFF** or **BOTH OFF** when you do not want the countdown timer alerts.

The Countdown time will remain set for future use.

*Note: The Green **COUNTDOWN TIMER indicator light** will not show on the display.*

CARE AND MAINTENANCE

“GLOBAL 360” has been carefully designed and manufactured to provide you with accuracy and dependability. The following suggestions will help you care for your clock/countdown timer so that you can enjoy the maximum from it.

Keep the clock dry. If it gets wet, wipe it dry immediately. Liquids can contain minerals that corrode the electronic circuits.

Use only a fresh battery of the recommended size and type. Always remove an old or weak battery. It can leak chemicals that can damage electronic circuits.

Handle the clock gently and carefully. Dropping it can damage the circuit boards and case and can cause it to work improperly.

Use and store the clock only in normal temperature environments. Extreme temperatures can shorten the life of electronic devices and distort or melt plastic parts.

Keep the clock away from dust and dirt, which can cause premature wear of parts.

Wipe the clock with a dampened cloth occasionally to keep it looking new. Do not use harsh chemicals, cleaning solvents, or strong detergents to clean the clock.

Modifying or tampering with internal components can cause a malfunction and will invalidate the warranty.

ONE YEAR LIMITED WARRANTY

Your “GLOBAL 360” is warranted to be free from defects in material or workmanship for One Year from the date of purchase. This is a limited warranty and covers functions only. The warranty is void if the units have been tampered with, altered, abused, or subjected to extreme conditions for which they were not designed, or repaired other than by a qualified service center. At its option, GLOBAL ASSISTIVE DEVICES, INC. may select to replace a unit, rather than repair it. During the warranty period, return the product, with dated proof of purchase, in a secure package, insured and with shipping charges prepaid. No C.O.D. packages will be accepted. Please include a note describing in detail the reason for return, your return address and include \$9.95 (check, money order, Visa, MasterCard, American Express, or Discover) for shipping and handling in the USA. Other Countries please contact us for pricing.

OUT OF WARRANTY SERVICE

Follow the same procedure for shipping to us as you would for warranty service. Be sure to include a description of the problem. An estimate will be submitted for your approval for any labor or parts not covered by the warranty before any work is performed.

DISCLAIMER:

In no event shall Global Assistive Devices, Inc. be held liable for any special, incidental, indirect or consequential damages whatsoever including, but not limited to, damages for loss of property, personal injury, loss of privacy, failure to meet any duty, negligence, and any other loss whatsoever, however caused, even if Global has been advised of the possibility of such damages. This disclaimer of liability applies to any damages or injury, including but not limited to those caused by any failure of performance of the product. Responsibility for selection, safety and suitability of any product rests with the consumer. Circumstances or consequences arising out of or in any way related to the use of or inability to use our products is the responsibility of the consumer. Global Assistive Devices, Inc. cannot be held liable in any way. When an accessory that is not manufactured by GLOBAL is used with this product, GLOBAL will not accept any liability for the performance or malfunction of the connected product or “GLOBAL 360”.

GLOBAL ASSISTIVE DEVICES, INC.
Oakland Park, FL USA 33334
www.GlobalAssistive.com

“Global 360” is a trademark of GLOBAL ASSISTIVE DEVICES, INC.