



Vibes Model VSS12 plugs into any clock or signaling/alerting device (with a 12 VDC/3.5mm output jack) to shake you awake.

The low voltage 3.5mm plug of *Vibes* plugs directly into the back of these units and when the alarm/signaler is activated, *Vibes* starts to vibrate.



The shape of *Vibes* allows for comfortable placement under a pillow. The strength of the vibrations is most sufficient for reliable use between the box spring and mattress.

Vibes Model VSS12 Bed Shaker has an automatic thermal shutoff with automatic reset. In the event that your signaling device or alarm is inadvertently left on, the thermal protector will stop the vibrations and then automatically reset when the unit is safe to start again.

GLOBAL ASSISTIVE DEVICES, INC.

Manufacturer of Assistive Devices for Special Needs

A

1121 East Commercial Boulevard, Oakland Park Florida USA 33334
Telephone: 954-776-1373 . Fax: 954-776-8136
E-mail: info@GlobalAssistive.com . www.globalassistive.com

- Your *Vibes* Model VSS12 Bed Shaker is warranted to be free from defects in material or workmanship for two years from date of purchase.
- This warranty does not apply to any unit which has been used for other than its intended use, or has been subjected to misuse, negligence or accident, or has been opened or serviced by anyone other than an authorized Service Depot of Global Assistive Devices, Inc.
- Global Assistive Devices, Inc. may select to replace the entire unit rather than repair it.
- For service, return the unit with proof of purchase date, in a secure package, insured, and with shipping charges prepaid. Please include a note describing the service required. Include your return street address. Enclose \$9.95 (check, money order, Visa, MasterCard, American Express, or Discover) for domestic shipping and handling. Other Countries please contact us for pricing.

GLOBAL ASSISTIVE DEVICES, INC.

Manufacturer of Assistive Devices for Special Needs

1121 East Commercial Boulevard, Oakland Park Florida USA 33334
Telephone: 954-776-1373 . Fax: 954-776-8136
E-mail: info@GlobalAssistive.com . www.globalassistive.com