

How *VibraLITE*® can help with:

POTTY TRAINING

Plan your Potty Schedule for Frequent Potty Breaks (i.e. 2 hours) and let “VibraLITE” do the reminding. When worn by the parent, you will not forget or need to be constantly watching the clock. When worn by the child, they know it is time to go each and every time the countdown reaches zero and vibrates on their wrist.

The Auto-Repeat Feature on the Countdown Timer will automatically reset and countdown again and again every 2 hours each time it reaches zero. These “interval” reminders are preset once and continue to repeat until you turn the countdown OFF with a push of a button. The Quick Reset Feature allows you to reset to the original time (i.e. 2 hours) with the push of a button. This is convenient when you need to have the child void prior to leaving home and want to then continue use the ongoing 2 hour reminders.

DISCLAIMER

GLOBAL ASSISTIVE DEVICES, INC. DOES NOT ACCEPT ANY RESPONSIBILITY WHATSOEVER FOR ANY INCONVENIENCE OR NONCOMPLIANCE FOR MEDICATION OR ANY OTHER PURPOSE THE “VibraLITE” WATCH IS USED FOR.

CONSULTATION WITH YOUR MEDICAL PROVIDER SHOULD ALWAYS BE DONE WHEN ATTEMPTING TO CONTROL OR TREAT ANY MEDICAL CONDITION.